

# Using the Roger™ Pass-around in the classroom

Reference Sheet

roger



The Roger Pass-around is designed to enhance classroom discussions so that not only teachers, but all students are heard clearly. The Roger Pass-around can be handed from one person to another or placed in the sturdy stand in front of one or two students.

## General operation

### General operation

- To switch on, press the On/Off button on the side of the Roger Pass-around for 3 seconds. The indicator lights will blink green as it powers up.
- To switch off, press the press the On/Off button on the side of the Roger Pass-around for 3 seconds. The indicator lights will blink green until powered off.
- When charging either with a USB power cord or in the Roger Charging Rack, the indicator light will blink green and when fully charged will shine solid green.



### Prior to first use

- Turn on the Roger Pass-around.
- Place the Roger Touchscreen Mic within 4" (10cm) of the Roger Pass-around and tap the **Connect** icon on the Touchscreen Mic.

### Daily use

The Roger Pass-around is fully automatic and ready to use once connected. The child just needs to talk into it to be heard.

- **Handheld** – The speaker just holds the mic near the mouth when speaking. The indicator light will shine green when the voice is picked up.
- **In the stand** – The Roger Pass-around can be placed in its sturdy yet light-weight stand. Place it on the desk in front of the speakers.
- If the Roger Pass-around is laid on the table, it will automatically mute (indicator light will blink red) to avoid broadcast of unnecessary sound.

## Optional operation

### Push-to-Talk

This is a specially programmed setting that requires the speaker to manually activate the Roger Pass-around to use.

- **Handheld** – the speaker must push and hold the large microphone button on the Pass-around. The indicator light will shine green when in use. All other Roger Pass-around indicator lights will be orange, indicating they are in standby.
- **In the stand** – The user must quickly press the large microphone button on the Roger Pass-around. The indicator light will turn green confirming use. When the speaker is finished, quickly press the button again to release the microphone.